DISTRIBÚTOR V SR:

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ENG desam® OX

Instruction for use: For disinfection purposes, it is diluted with water at a temperature of 20 °C – 25 °C, to make a concentration of the solution shown in the table. Visibly dirty surfaces and items are at first cleaned mechanically and afterwards disinfected. Not suitable for galvanized surfaces, non-ferrous metals (copper, brass), polyamide, leather, rubber, color unstable, porous and damaged materials. desam® OX is not suitable for disinfection of invasive medical devices. **Active substances:** Hydrogen peroxide (110 g/kg) (EC 231-765-0), Didecyldimethylammonium chloride (20 g/kg) (EC 230-525-2), Alkylbenzyltrimethylammonium chloride (28 g/kg) (EC 270-325-2). **Auxiliary ingredients:** Guanidine, N,N"-1,3-propanediylbis-, N-coco alkyl derivs., diacetates; alcohol (EO 8) ethoxylated C10 **Attention:** For professional use only. **Danger H314** Causes severe skin burns and eye damage. **H410** Very toxic to aquatic life with long-lasting effects. **P273** Avoid release to the environment. **P280** Wear protective gloves with nitrile coating – chemically resistant / protective clothing / eye protection / face protection. **P301+P330+P331** IF SWALLOWED: rinse mouth. Do NOT induce vomiting. **P303+P361+P353** IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water [or shower]. **P305+P351+P338** IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **P310** Immediately call a doctor. **First aid: Inhalation:** Shut off source of exposure, if possible. Bring the victim to the fresh air, keep at rest (avoid even walking if necessary, seek medical attention). Any serious incidents related to the product are to be reported to the manufacturer and competent authority. **Storage and safe disposal of the containers:** Store in the original containers in dry and clean rooms separately from food, drinks and feed. Do not store near the source of heat and in direct sun light. Storage temperature is in the range from -10 °C to +25 °C. Residual amount of the product to be applied in accordance with the directions for use, bigger amount to be transmitted to authorised companies for liquidation. Contaminated packaging hand over to specialised company as a dangerous waste. **Durability:** Use before date on the packaging.