**DE Kalt/Warm Maske/Migräne-Brille**

**GB Cold/warm mask**

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**IT Freddo/caldo maschera**

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**PT Máscara quente/frio**

**CZ Studená/teplá maska**

**SK Maska chlad/teplo**

**REF 93.600.00.100**

PZN - 8825869

(01)04034342074043

LOT

P. J. Dahlhausen & Co. GmbH  
Emil-Hoffmann-Straße 53  
50996 Köln · Germany  
Tel.: +49 (2236) 3913-0  
Fax: +49 (2236) 3913-109  
info@dahlhausen.de  
www.dahlhausen.de

Rev. 00/2012-06

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## GB INSTRUCTIONS FOR USE

### Instructions for use Cold/warm mask:

The Cold/warm mask is a re-usable facial and cosmetic mask for use with facial pain and to support cosmetic treatments.

### Fields of use COLD APPLICATION:

Headaches and migraine. Tired and swollen eyes. Fever, hangover.

### Fields of use WARM APPLICATION:

Colds, influenza, sinusitis. To support cosmetic treatments, for example to stimulate the circulation.

### Contraindication:

Do not use the mask if the following conditions are present: Compromised skin feeling and disturbed sensitivity of the affected area (e.g. diabetes, children under 3 years of age, persons sensitive to heat/cold).

### Instructions for use COLD APPLICATION:

Place the mask in the fridge for at least 2 hours.

### Instructions for use WARM APPLICATION:

Lay the mask flat in the microwave, distribute the gel evenly. Heat for 1 minute at 300 Watts (thaw setting). Remove the mask and knead it well to distribute the warmth equally. If the desired temperature has not yet been reached, repeat the process in 20 second intervals.

### Usage instructions:

Never freeze the mask. Do not warm up in the oven or grill. Overheating can damage the mask. Before application, test the temperature of the mask on forearm. Do not leave microwave unattended. Immediately interrupt the heating process if the mask inflates. Leaky or empty masks may no longer be used. Danger of burning!

### Information on care and storage:

In case of soiling, wipe off the mask with a damp cloth. Store the mask in the fridge.

### Information on the gel and on disposal:

The gel contained in the mask is non toxic. If gel should contact the skin or eyes, simply wash the area extensively with water. Irritations of skin and eyes are not known to occur. Dispose of the mask in the household waste (residual waste).

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